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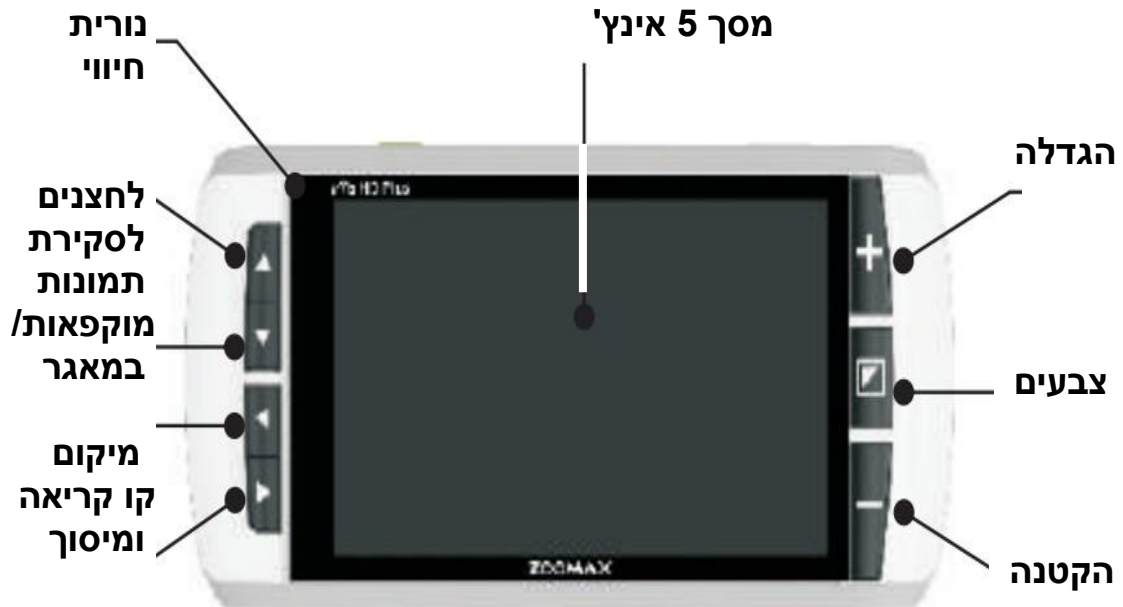
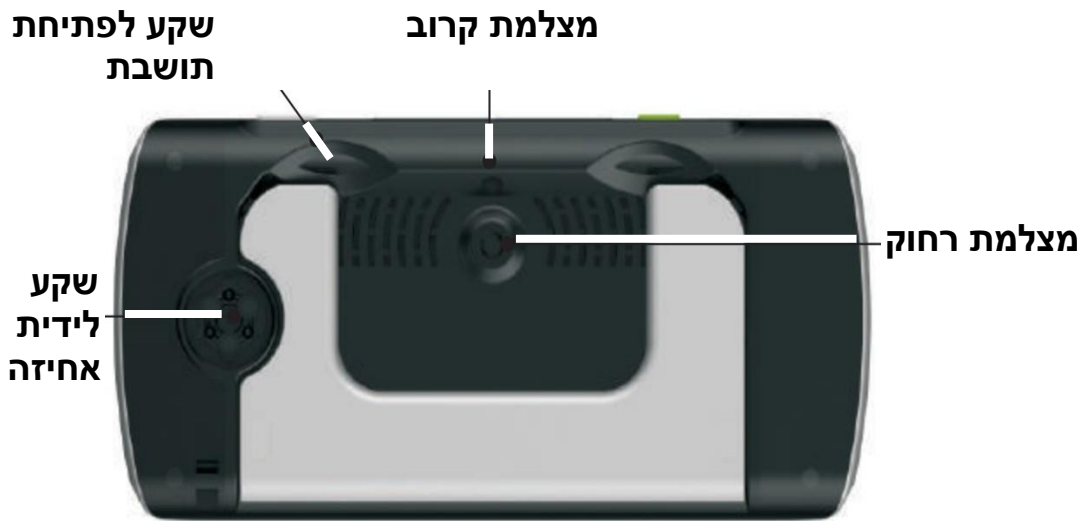
	Function	Operation
	Turn on/off the device	Long press  for 3 seconds
	Zoom in/ out	Press  or 
	Change color mode	Short press 
	Favorite color mode setting	To set the favorite color, long press  for 3 seconds
	Tap to focus	When using the distance-view function, tap the screen to focus
	Freeze image	Short press 
	Save photo	Press  and  simultaneously for 3 seconds
	Check saved photo	Long press  for 3 seconds

	Function	Operation
	Select saved photo	Press  or  to browse, short press  button to confirm Under the lowest magnification, press  or  to view the previous or next photo
	View saved/ frozen photo	Press  or  to adjust magnification; when the photo is zoomed in, press the arrow buttons     or press the screen and pan to view details
	Delete saved photo	Select the photo you want to delete, then long press  for 5 seconds, the photo will be deleted automatically
	Revert to real-time viewing mode after photo operation	Short press 

	Function	Operation
	Demo mode	Long press  for 5 seconds to enter or exit the demo mode
	Sound on/off	Press  and  simultaneously for 3 seconds to enter the setting menu.
	LED on/off	Press  or  to select the mode you want to change, then
	Frame rate setting	short press  to rotate and change
	Brightness adjustment	Long press  and  simultaneously for 3 seconds to enter the setting menu. Press  or  to select the brightness icon. Press  to confirm and press  or  to choose the brightness level

	Function	Operation
	Reading line & Mask	<p>Short press  and  simultaneously to cycle through reading line and reading mask</p> <p>Press     to adjust the position of reading line or mask</p>